




THE WORKHORSE SCLERAL LENS

Practitioners around the country are reaching for Ampleye as their scleral lens of choice for its simple, straightforward design and powerful on-eye performance. The intuitive design is easy to learn for beginning fitters, while offering the versatility required by experienced practitioners in an established specialty lens practice.

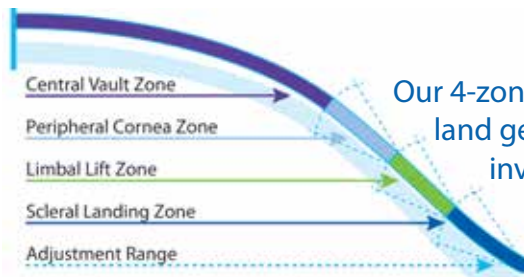

Our 4-zone lens is designed to fully vault the cornea and limbus and land gently on the sclera with 360° of alignment. With no corneal involvement, irregular cornea fitting is simplified, and patient comfort and acuity are optimized.

EFFICIENT DIAGNOSTIC FITTING SAVES CHAIR TIME

The intelligently designed Ampleye diagnostic set includes the optimal number of lenses to fit all types of patients - from a highly prolate bulging graft to a low sag oblate cornea.

The incorporated 150 micron toric haptic provides rotational stability, ensures accurate trial assessment, and eliminates guesswork on Rx orders. The consistent Limbal Lift and Scleral Landing Zones mean that only one Dx lens application is required for on-eye assessment, reducing stress for the patient and saving valuable chair time for the practice.



chair time
with other
sclerals ++



OUR GOAL: HIGHLY SATISFIED PATIENTS, A FASTER SUCCESSFUL ENDPOINT

✉ info@artoptical.com

☎ 800.253.9364

🌐 www.artoptical.com

ARTOptical
contact lens, inc.

- 16.5mm optimal diameter (15.0-17.0mm available)
- Independent zone adjustments available for infinite control of the lens fit
- Optimally engineered landing zone for wide & even distribution of pressure, minimizing blanching & edge modifications
- Employing Spline Science for an incredibly forgiving fit & enhanced first-fit success
- Advanced aberration correcting optics insure a precise power profile throughout the optical zone
- Aided by inherent toricity, the full circumference closure ensures soothing moisture retention for lasting hydration and comfort and minimal lens fogging due to trapped debris

DESIGN DETAILS & PARAMETERS

SPHERE POWER

+/-20.00* in 0.25D steps

*Can be extended on order of practitioner

CYLINDER POWER Up to -5.00D in -.25 steps

AXIS 1° to 180° in 1° steps

THICKNESS

Standard = .30mm; Adjustable in .10 steps to .50mm

ZONE CONTROL ADJUSTMENTS

Peripheral Cornea Zone (PCZ)

Limbal Lift Zone (LLZ)

Scleral Landing Zone (SLZ)

Adjustable in 1.0 steps to +/-15.00

Each step = 25µm of change

SLZ QUADRANT CONTROL

Q1 & Q3 = 0 | Q2 & Q4 = 150µm

Quadrants adjustable in 1.0 steps to +/- 15.00

CENTER NEAR MULTIFOCAL

Add Power: +1.00D to +3.50D in 0.25D steps

Standard center near zone = 2.50mm; adjustable from 1.00-4.00mm in 0.50mm steps; Custom Aligned Optics insure appropriate near zone positioning

FEATURED MATERIALS

OPTIMUM
BY Contamac®

PATIENT CONSIDERATIONS

Ampleye is an excellent option for patients with:

- Irregular Corneas
- Lens Stability Issues
- Corneal GP Intolerance
- Dry Eye & Ocular Surface Disease

EXPERT TECHNICAL SUPPORT

Our large, experienced professional consultation team is available weekdays from 9:00 am- 6:00 pm EST to enhance your Ampleye fitting experience.

IMMEDIATE AVAILABILITY

Ampleye lenses are manufactured and shipped in just 24 -48 hours!

ONE WARRANTY, NO WORRIES

Unlimited exchanges! No lens returns required*! No material exchange fees! Our worry-free fitting warranty has you covered for 120 days from the initial order date.

*Exceptions apply for cancellation and non-parameter or Rx related exchanges.

FREE PATIENT RESOURCE KIT

Included with each Rx lens order!

Available Add-On Options

- No-Worry Warranty
- Front Toric
- Multifocal
- Quad Control
- Edge Vault
- HOA Correction
- tangible Hydra-PEG

PROVIDE PEACE OF MIND

Plan ahead to offer a spare lens option to your Ampleye patients! Duplicate non-warranted Ampleye lenses may be purchased at 25% off within 90 days of order.



Be sure to request tangible Hydra-PEG treatment for enhanced surface properties & increased wearing time!